

RapidVet® -H Crossmatch Centrifuge Settings

NOTE: ALL ARE TO BE RUN AT FULL SPEED UNLESS OTHERWISE NOTED

MAKE	MODEL		RPM	RCF (GX)	MINUTES
APEXX	CENTRI-V1586		3300	4060	1.6
BECKMAN	GS6		6000	5200	1.25
CLAY ADAMS	COMPAC II		3200	1315	5
CLAY ADAMS	DYNAC		3000		5
CLAY ADAMS	PHYS COMPACT		3400		5
CLAY ADAMS	SEROFUGE		3400	1049	6.2
CLAY ADAMS	TRIAC	BLOOD	3500	1546	4.16
CLAY ADAMS	TRIAC	SERUM	3800		7
CLAY ADAMS	TRIAC multifunction		10,400	12,600	0.5
DENVILLE SCIENTIFIC	260D		14,000	16,000	0.5
DRUCKER	613B		3100	1240	5
FISHER SCIENTIFIC	COMPAC 10V		10,000	5600	1.16
FISHER SCIENTIFIC	228		3,300	1380	4.75
HAMILTON BELL	1505		3,400	1380	5
HORIZON	HORIZONTAL		3380	1600	4
HORIZON	FIXED ANGLE		3450	1450	5
JORGENSEN	ULTRA 8		3300	1327	5
JORGENSEN	J502M		4000	2415	2.69
JORVET	J501		3300	1327	5
LEADING EDGE	CENTRI-V 1586		5000	4100	1.5
LW SCIENTIFIC	815		3200	1219	5.3
LW SCIENTIFIC	C3	BLOOD	3000	1500	4.33
LW SCIENTIFIC	C5		5000	4000	1.5
LW SCIENTIFIC	STRAIGHT 8		5000	4000	1.5
LW SCIENTIFIC	STRAIGHT 8		3000	1500	4
LW SCIENTIFIC	ULTRA 8V		3300	1350	5
LW SCIENTIFIC	ULTRA 8F		3300	1350	5
LW SCIENTIFIC	ULTRA 8 SELECT		3300	1350	5
LW SCIENTIFIC	ULTRA 8TL		3300	1350	5
LW SCIENTIFIC	V24 COMBO	LOW	4000	2325	2.8
LW SCIENTIFIC	ZIPOCRIT		11000	4400	1.5
LW SCIENTIFIC	ZIPSPIN		7000	2800	2.5
STAT SPIN	EXPRESS 2	NORMAL	8500	4440	1.5
STAT SPIN	MP	URINE X 2	9800	3900	1.5
STAT SPIN (IDEXX)	VT		15800	13700	0.5
UNICO	POWERSPIN		3400	1350	5
UNICO	POWERSPIN FX	C806 & C808	3400	1320	5
UNICO	POWERSPIN LX C856		3100	1097	6
UNICO	POWERSPIN LX C858		4000	1828	3.5
UNICO	POWERSPIN MX		3400	1825	3.5
VULCON	CLINASPIN & CLINASEAL		3150	1132	5
VANGUARD (Hamilton Bell)	V6500		3400	1318	5
VARISEAL	VS6C & VS8C		3150	1132	5.75
VWR	GALAXY MINI or MINISTAR		6000	2000	3.25
VWR	GALAXY ARRAY		4000	1216	5.35
WOODLEY/DRUCKER	CLINISPIN MICRO XM		7000		2.5
WOODLEY/DRUCKER	CLINISPIN HORIZON 842VET		8000		1
WOODLEY/DRUCKER	CLINISPIN HORIZON 642E		3380	1600	4
WOODLEY/DRUCKER	CLINISPIN 1000HS		8000		1

AS OF 10/17/11

dmslaboratories, inc.

www.rapidvet.com Tel: 800-567-4367