

RapidVet[®]-H

Canine DAL Blood Typing

Canine Interpretation Guide

Examples of Agglutination



1+



2+



3+



4+

If the assay was run correctly, visible agglutination should have occurred in the well marked “Positive Control.”

If the patient sample shows visible agglutination (see examples of positive agglutination above) in the well marked “Patient Test,” the patient is DAL positive. The “Auto-Agglutination Saline Screen” well should not show agglutination. (Auto-Agglutination is a rare occurrence.)

Negative for Agglutination



If no agglutination is visible in the well marked “Patient Test,” (see negative example above), the patient is DAL negative.

dmslaboratories, inc.

2 Darts Mill Road, Flemington, NJ 08822

Tel.: (908) 782-3353 or (800) 567-4367 in U.S. and Canada

E-mail: dms@rapidvet.com

RapidVet is a registered trademark of **dms**laboratories, inc.