

Rapid Vet-H

Interpretation Guide - DEA 4

Examples of Positive Agglutination



Example 1



Example 2



Example 3



Example 4

If the assay was run correctly, visible agglutination should have occurred in the well marked "Positive Control."

If the patient sample shows visible agglutination (see examples of various degrees of positive agglutination above) in the well marked "Patient Test," the patient is DEA 4 positive. The "Auto-Agglutination Pre-Screen" well should not show agglutination. (Auto-Agglutination is a rare occurrence.)

Negative for Agglutination



If no agglutination is visible in the well marked "Patient Test," (see negative example above), the patient is DEA 4 negative.

dmslaboratories, inc.

2 Darts Mill Road, Flemington, NJ 08822

Tel.: (908) 782-3353 or (800) 567-4367 in U.S. and Canada

E-mail: dms@rapidvet.com

RapidVet is a registered trademark of **dms**laboratories, inc.