

## Interpretation Guide - DEA 5

### Examples of Positive Agglutination




Example 1



Example 2



Example 3

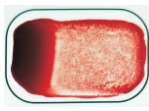

Example 4

If the assay was run correctly, visible agglutination should have occurred in the well marked "Positive Control."

If the patient sample shows visible agglutination (see examples above) in the well marked "Patient Test," the patient is DEA 5 positive. The "Auto-Agglutination Pre-Screen" well should not show agglutination. (Auto-Agglutination is a rare occurrence.)

Note: DEA 5 positive reactions will not be as strong as DEA 1 positive reactions.

### Negative for Agglutination



If no agglutination is visible in the well marked "Patient Test," (see negative example above), the patient is DEA 5 negative.

Photos may be viewed online at <http://www.rapidvet.com/downloads.html>

**dms**laboratories, inc.

2 Darts Mill Road, Flemington, NJ 08822

Tel.: (908) 782-3353 or (800) 567-4367 in U.S. and Canada

E-mail: [dms@rapidvet.com](mailto:dms@rapidvet.com)

RapidVet is a registered trademark of **dms**laboratories, inc.