






RapidVet[®]-D

Dermatophyte Screening

<p><u>Day 1:</u> Step 1 → Temperature</p>  <p>Bring tube to room temperature 72-77°F BEFORE inoculating.</p>	<p>Step 2 → Sampling</p>  <p>Take sample from EDGE of legion. Use only hair root (bottom 2cm) as sample.</p>	<p>Step 3 → Inoculation</p>  <p>Place sample on SURFACE of substrate.</p>	<p>Step 4 → Capping</p>  <p>Cap tube LOOSELY. Let stand at 72-77°F.</p>	<p><u>Days 2-4:</u> Step 5 Results</p>  <p>Any color change to RED is a Positive result.</p>
---	---	--	--	---