


# RapidVet<sup>®</sup>-H (Feline A, B & AB) Instructions for Use


Dispense 1 drop of Diluent into the Auto-Agglutination well. Dispense 1 drop of patient blood into the same well. Stir well for 10 seconds using a stirrer.

## Auto-Agglutination Pre-Screen

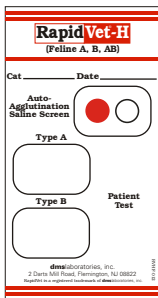


Dispense 1 drop of Diluent into each remaining well.

## Step 1:

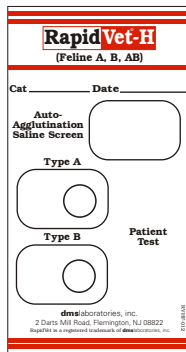


Dispense 1 drop of patient blood into the well marked Type A and dispense one drop of patient blood into the well marked Type B

## Step 2:

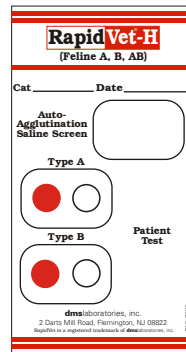

Stir Type A well with a stirrer for 10 seconds. Using a NEW stirrer, stir the Type B well for 10 seconds.

## Step 3:

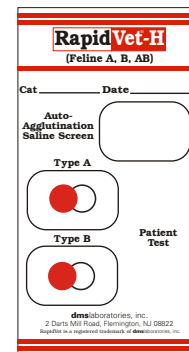


If no agglutination is observed in the well, then you may proceed with the test.

Add a second drop of diluent to the well marked Type A. Do not stir the well again with a stirrer.


Pick the card up and rock and tilt by wrist action for 2 minutes.

## Step 5:

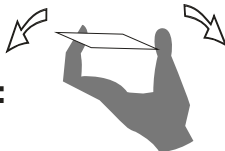

Position the card at a 10° - 20° angle.


## Step 6:

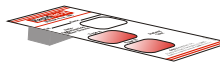

Read the results:

## Step 7:

**See Photo Interpretation Guide**

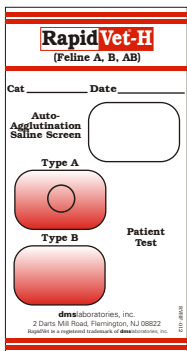

## Step 4: