


RapidVet[®]-H


Companion Animal Crossmatch

Crossmatch Photo Identifier

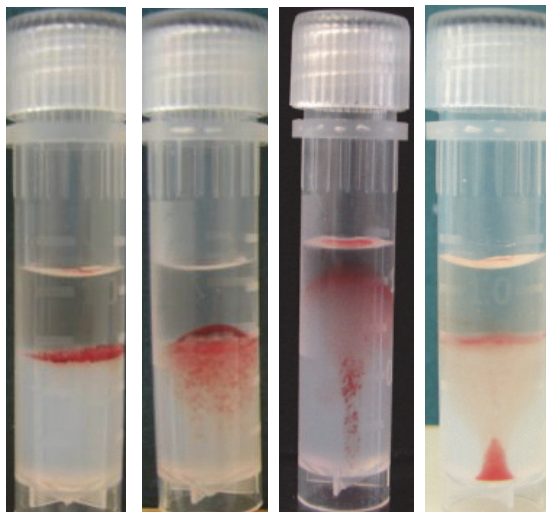

POSITIVE (Incompatible donor/recipient)

A positive test is considered any tube demonstrating red blood cells at or near the top of the gel column after centrifugation. In weak agglutinations, the cells may end up at the bottom of the tube in a conical or bell-shaped configuration. In this instance, the peak of the reaction will extend more than halfway up the gel column. **DO NOT TRANSFUSE USING THIS DONOR.**

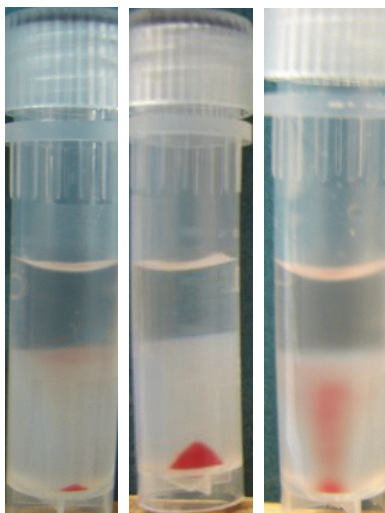

NEGATIVE (Compatible donor/recipient)

A negative test is considered any tube where the vast majority of red blood cells are at or near the bottom of the gel column after centrifugation and no firm line of agglutinated cells remains at the top of the gel. **IT IS UNLIKELY THAT A SIGNIFICANT REACTION WILL RESULT FROM A TRANSFUSION USING THIS DONOR.**

Rev. 11/2024

dmslaboratories, inc.

3252 South Fair Lane, Tempe, AZ 85282

Tel.: (602) 437-5181 or (800) 567-4367 in U.S. and Canada

www.rapidvet.com dms@rapidvet.com

RapidVet is a registered trademark of **dms**laboratories, inc.